

DRY ICE specimen shipment preparation

- Samples will be shipped in dry ice
- All primary and back-up samples must be send separately
- All samples has to be send to Synevo Central Lab once a month during enrollment period



- All kinds of samples from each visit must be shipped in separate plastic bags

- All primary and back-up samples must be shipped in special boxes



Synevo Central Lab will inform the sites about the exact shipment dates!

- Place special boxes and plastic bags with specimens in the middle of container containing dry ice



- Place the foil bag with Requisition Forms and Sample Shipment Inventory List at the top of the container with dry ice
- Close carton box
- Box should be sent according to shipping section in this manual to the Synevo Central Lab