

URINALYSIS (URINE)


First morning sample of urine has to be taken from the urine container.

It is recommended to perform the test on the first morning urine sample but if this is not possible a later urine sample can be used.

Before taking urine sample patient should thoroughly clean the genital area.

STEP 1

Fill the proper container with the first morning sample of urine.
Mix carefully the container with urine sample.

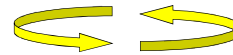

STEP 2

Put the tube of the transfer unit into the urine container and press the urine tube (yellow/red stopper) into the top. Wait until the urine fill the tube.



STEP 3

Mix carefully the tube with the urine sample. Store in the refrigerator (2-8°C) until shipment to Synevo Laboratory.



STEP 4

SHIP REFRIGERATED according to the "Specimen shipment preparation" and "Shipping" section to this manual.